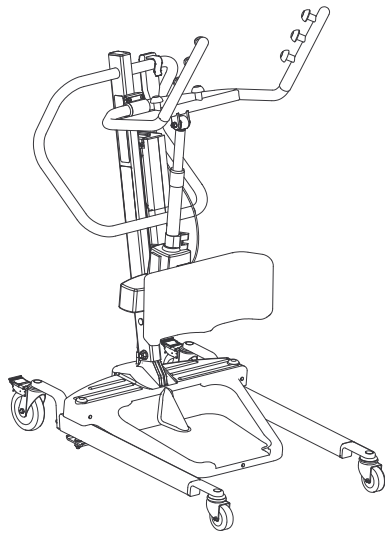
Invacare® Stand Assist

RPS350-1E, RPS350-1FR

en	Patient Lift User Manual	3
de	Aufstehlifter Gebrauchsanweisung	45
es	Grúa de bipedestación Manual del usuario	95
fi	Potilasnostin Käyttöohje	137
fr	Lève-personne verticalisateur Manuel d'utilisation	181
it	Sollevatore paziente Manuale d'uso	228
nl	Stand Assist Patiëntlift Gebruiksaanwijzing	273
pt	Elevador de paciente Manual de utilização	318

This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

© 2017 Invacare Corporation.

All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted. A spare parts catalog is available upon request.

Contents

This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

1 General	4
1.1 Symbols	4
1.2 Intended Use	5
1.2.1 Contraindications	5
1.3 Included Items	6
1.4 Service Life	7
1.5 Warranty Information	7
2 Safety	8
2.1 General Guidelines	8
2.2 Operating Information	8
2.2.1 General	9
2.2.2 Positioning	10
2.2.3 Disposal	10
2.3 Radio Frequency Interference	11
2.4 Product Labeling	12
3 Setup	13
3.1 Safe Assembly	13
3.2 Assembling the Mast to the Base	14
3.3 Preparing the Lift for Use	14
3.4 Installing the Shifter Handle	15
3.5 Attaching the Battery Charger Mounting Bracket to the Wall	15
4 Usage	17
4.1 Introduction	17
4.2 Closing/Opening Legs	17
4.2.1 Closing/Opening Manual Legs	17
4.3 Locking/Unlocking the Rear Casters	18
4.4 Raising/Lowering the Lift	18

4.4.1 Raising/Lowering an Electric Lift	18
4.5 Activating a Mechanical Emergency Release	19
4.5.1 Primary Emergency Release	19
4.5.2 Secondary Emergency Release	19
4.6 Performing an Emergency Stop	20
4.7 Charging the Battery	20
4.7.1 Battery Indicator	20
4.7.2 Using a Power Cord to Charge the Battery	22
4.7.3 Using the Battery Charger to Charge the Battery	23
5 Lifting the Patient	24
5.1 Safe Lifting	24
5.2 Positioning the Stand Assist Lift	26
5.3 Lifting the Patient	27
5.4 Moving the Patient	29
6 Transferring the Patient	30
6.1 Safe Transfer	30
6.2 Transferring to a Commode	31
6.3 Transferring to a Wheelchair	33
6.4 Transferring to a Bed	34
7 Troubleshooting	35
7.1 Troubleshooting Table	35
8 Maintenance	37
8.1 Maintenance Safety Inspection Checklist	37
8.2 Cleaning the Sling and the Lift	38
8.3 Reuse	39
8.4 Detecting Wear and Damage	39
8.5 Lubricating the Lift	39
8.6 Adjusting the Base	40
8.7 Replacing the Electric Actuator	41
8.8 Adjusting the Knee Pad Height	41
9 Technical data	43
9.1 RPS3501E and RPS3501FR Stand Assist Patient Lift	43

1 General

1.1 Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING!

– Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION!

– Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.

IMPORTANT

– Indicates a hazardous situation that could result in damage to property if it is not avoided.

Gives useful tips, recommendations and information for efficient, trouble-free use.

Date of manufacture.

This product complies with Directive 93/42/EEC concerning medical devices.

The launch date of this product is stated in the CE declaration of conformity.

Read Manual

Audible tone when battery low. Refer to 4.7 Charging the Battery, page 20.

Caster lock.

Open/close Legs

Raise/lower the lift arms

Emergency stop

Safe Working Load

Double Insulated, Class II equipment

Type B applied part

Recycle this product. Refer 2.2.3 Disposal, page 10.

1.2 Intended Use

WARNING!

Risk of Falling

The Invacare stand assist lift is NOT a transport device. It is intended to transfer an individual from one resting surface to another (such as a bed to a wheelchair).

Invacare slings and patient lift accessories are specifically designed to be used in conjunction with Invacare patient lifts.

To avoid injury or falls:

- Patients who are transferred and positioned using the stand assist lift **MUST** be cooperative, coherent and have head and neck control. Otherwise, injury may occur.
- Patients who are transferred and positioned using the stand assist sling **MUST** be able to support a majority of their own weight. Otherwise, injury or damage may occur. Refer to the sling user manual for more information.

Stand assist lifts are battery-powered transfer devices, designed to be used in most of the common lifting situations, for example:

- Between the bed and a wheelchair

- To and from the toilet

The stand assist lift is only intended to lift patients up to the maximum weight limit stated in technical data.

Selecting the appropriate slings and accessories for each individual is important to assure safety when using a patient lift. See Invacare's sling and accessory user manuals for further information on those devices.

Invacare recommends that the patient be transferred to a shower chair or other means for bathing.

The stand assist lift can be turned (rotated) in place for transfers with limited floor space.

1.2.1 Contraindications

The stand assist patient lift is contraindicated for patients who:


- are not cooperative
- are not coherent
- do not have head and neck control

Some slings designed for the stand assist patient lift are contraindicated for patients who cannot support the majority of their weight. Refer to the sling user manual for more information.

1.3 Included Items

The items listed in the tables are included in your package. Slings are sold separately. The wall charger may be sold separately.

Item	Description	Quantity
A	Lifter	1
B	Battery	1
C	Charging Cord, EU	1
D	Charging Cord, UK (RPS350-1E only)	1
E	Hand Pendant	1
F	Lifter User Manual (not shown)	1

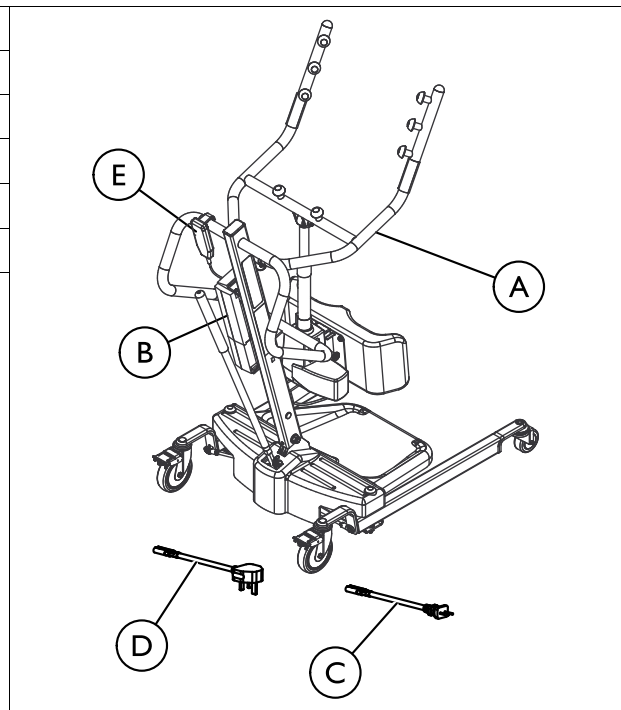1.4 Service Life

WARNING!

Risk of Injury or Damage

The product has been tested for the service life stated in this manual. Use of the product beyond this time period may cause product damage and injury.

- Only use the product for the service life stated in this manual. DO NOT exceed the service life of the product.
- Remove the product from service when the service life has been met. DO NOT continue to use the product.
- Perform all maintenance according to the recommended schedule in this manual.

The mobile patient lift has an expected lifetime of 8 years when used in accordance with safety instructions, maintenance intervals and correct use, stated in this manual. The effective product lifetime can vary according to frequency and intensity of use.

Actuator Service Life

Number of lifts per day	Service life of actuator (in years)
1–2	10
3	9
4	6
5	5

Number of lifts per day	Service life of actuator (in years)
6	4
7–9	3
10–13	2
14–27	1

1.5 Warranty Information

Terms and conditions of the warranty are part of the general terms and conditions particular to the individual countries in which this product is sold.

Contact information for your local Invacare office is located inside the back cover of this manual.

2 Safety

2.1 General Guidelines

**WARNING!**

– DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

ACCESSORIES WARNING

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories.

Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products. In certain instances, the use of other manufacturer's slings may be possible.

– Contact your local Invacare office for more information about accessories.

**NOTICE**

– The information contained in this document is subject to change without notice.

Check all parts for shipping damage before using. In case of damage, DO NOT use the equipment. Contact the Dealer or Invacare representative for further instructions.

2.2 Operating Information

This section of the manual contains general safety information about your product. For specific safety information, refer to the appropriate section of the manual and procedures within that section. For instance, for safety information related to assembling the lift, refer to the Setup section of the manual.

2.2.1 General

WARNING!
Risk of Falling

DO NOT attempt any transfer without approval of the patient's physician, nurse or medical assistant. Thoroughly read the instructions in this user manual, observe a trained team of experts perform the lifting procedures and then perform the entire lift procedure several times with proper supervision and a capable individual acting as a patient.

- Use common sense in all lifts. Special care MUST BE taken with people with disabilities who cannot cooperate while being lifted.
- Use steering handle on the mast at all times to push or pull the patient lift.
- Be sure to check the sling attachments each time the sling is removed and replaced, to ensure that it is properly attached before the patient is removed from a stationary object (bed, chair or commode).

WARNING!
Risk of Injury or Damage

The patient lift can be used indoors or outdoors. Some surfaces may cause the lift to be unstable and cause injury or damage.

The lift can be used in the shower or bath area, but excessive moisture will damage the lift and may cause injury.

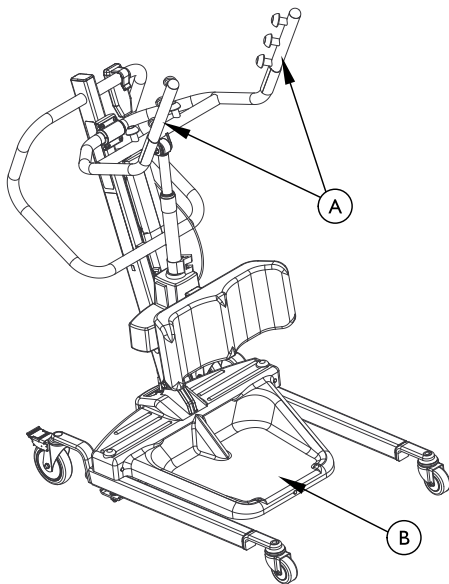
- Avoid using the lift on an incline. Invacare recommends that the lift only be used on a flat surface.
- DO NOT roll casterbase over uneven surfaces that may cause the patient lift to tip over.
- DO NOT use the patient lift in the shower or bath or in any prolonged moisture environment.
- Ensure that the patient lift is wiped clean of any moisture after use.
- DO NOT store the lift in a damp area or in a damp condition. Refer to the storage temperature, humidity and pressure ranges listed in 7.1 Troubleshooting Table, page 35.
- Periodically inspect all components of the patient lift for signs of corrosion. Replace all parts that are corroded or damaged.

2.2.2 Positioning

WARNING! **Risk of Injury**

- ALWAYS be aware of the Lift Arms. Injury to the patient and/or assistant may occur.
- ALWAYS be aware of the Footrest, especially the patient's position on the footrest. Injury to the patient and/or assistant may occur.

Ⓐ = Lift Arms

Ⓑ = Foot plate

2.2.3 Disposal

WARNING! **Environmental Hazard**

This product has been supplied from an environmentally aware manufacturer that complies with the Waste Electrical and Electronic Equipment (WEEE) Directive 2012/19/EU. Device contains lead acid batteries.

This product may contain substances that could be harmful to the environment if disposed of in places (landfills) that are not appropriate according to legislation.

- DO NOT dispose of batteries in normal household waste. They MUST be taken to a proper disposal site. Contact your local waste management company for information.
- Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

2.3 Radio Frequency Interference

WARNING!

Risk of Injury or Damage

Most electronic equipment is influenced by Radio Frequency Interference (RFI).

CAUTION should be exercised with regard to the use of portable communication equipment in the area around such equipment, otherwise injury or damage may occur.

If RFI causes erratic behavior:

- PUSH the Red Power Switch OFF IMMEDIATELY.
- DO NOT turn the Power Switch ON while transmission is in progress.

3 Setup

3.1 Safe Assembly

**WARNING!****Risk of Injury**

- Improper assembly may cause injury or damage.
- Assembly **MUST** be performed only by qualified personnel.
 - Use only Invacare parts in the assembly of this patient lift. The lift components are manufactured to specifications that assure correct alignment of all parts for safe functional operation.
 - **DO NOT** overtighten the mounting hardware. This will damage the mounting bracket.

There are no tools required to assemble the patient lift.

If there are any issues or questions during assembly, contact a local Invacare representative. Refer to the contact information in the back of this manual.

3.2 Assembling the Mast to the Base

WARNING!

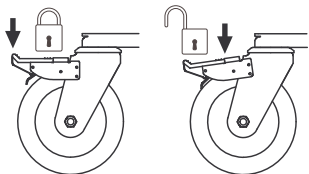
– Use only Invacare parts in the assembly of this patient lift. The base legs, the mast, boom, pump assembly and swivel bar are manufactured to specifications that assure correct alignment of all parts for safe functional operation.

3. Remove the hex bolt, washers and nut that are located in the U-shape cut-out of the base Refer to Detail B.
4. Lift the mast to an upright position.
5. Lower the mast onto the mounting bracket.
6. Attach the mast to the base with the hex bolt, washers and nut. Tighten securely.


3.3 Preparing the Lift for Use

Check and tighten all hardware BEFORE use.

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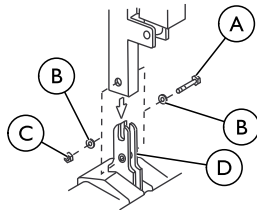The mast assembly may be removed from the base for storage or transporting. The mast assembly **MUST** be properly secured to the base assembly before use.

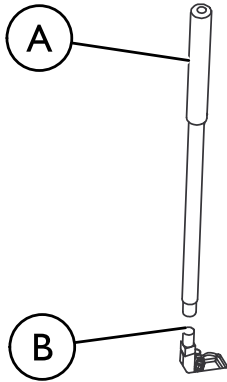
1. Put the base on the floor.

Make sure all four casters make contact with the floor.

2. Lock both rear casters Refer to Detail A.

3.4 Installing the Shifter Handle


Ⓐ Shifter Handle

Ⓑ Male Adapter in Base

1. Remove the shifter handle from the packaging carton.
2. Thread the shifter handle onto the male adapter on the base.

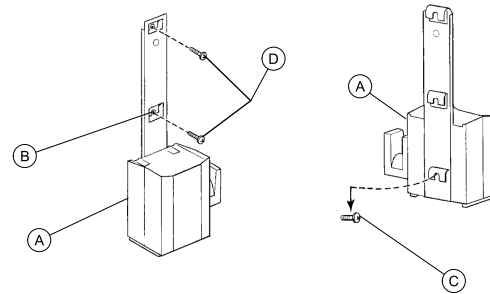3.5 Attaching the Battery Charger Mounting Bracket to the Wall

Refer to your local regulations concerning proper mounting procedures.

This procedure only applies to RPS350–1E models.

1. Place the battery charger with mounting bracket Ⓐ on the wall at the desired position.
2. With a pencil, mark the middle hole Ⓑ position.
3. Measure down 6½ inches (16.5 cm) from the pencil mark and drill one mounting hole.
4. Install the bottom mounting screw Ⓒ until there is an approximate 1/8-inch (3 mm) gap between the screw head and the wall.
5. Install the battery charger with mounting bracket onto the bottom mounting screw.
6. Drill the remaining two mounting holes.

7. Install the two remaining mounting screws ④ through the mounting bracket and into the wall. Tighten securely.
8. Plug the battery charger into the wall electrical outlet.
 ON LED should illuminate.

4 Usage

4.1 Introduction

The operation of the patient lift is an easy and safe procedure.

 Before using the lift with a patient, refer to the following procedures for safety information and instruction:

- 2.2 Operating Information, page 8
- 5.3 Lifting the Patient, page 27

4.2 Closing/Opening Legs

WARNING! **Risk of Injury**

The lift could tip and endanger the patient and assistants.

- The legs of the lift must be in the maximum open position for optimum stability and safety. If it is necessary to close the legs of the lift to maneuver the lift under a bed, close the legs of the lift only as long as it takes to position the lift over the patient and lift the patient off the surface of the bed. When the legs of the lift are no longer under the bed, return the legs of the lift to the maximum open position.

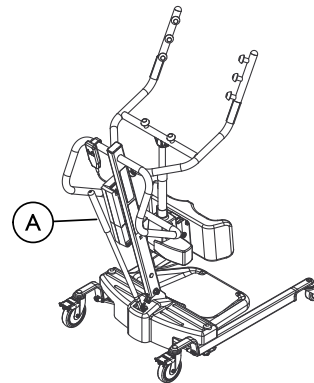
4.2.1 Closing/Opening Manual Legs

The shifter handle is used to open or close the legs of the base for stability when lifting a patient.

Refer to the safety information in 4.2 Closing/Opening Legs, page 17 before performing this procedure.

 = Shifter Handle

1. To close the legs:
Pull the shifter handle **OUT** and away from the stand-up lift and then to your **LEFT** until it **LOCKS** in the notch of the bracket.

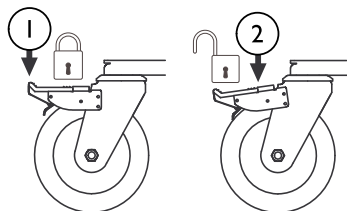
Left is determined by standing behind the stand-up lift facing towards the front casters.

2. To open the legs:
Pull the shifter handle **OUT** and away from the stand-up lift and then to your **RIGHT** until it **LOCKS** in the notch of the bracket.

Right is determined by standing behind the stand-up lift facing towards the front casters.

4.3 Locking/Unlocking the Rear Casters

1. Step here to lock
2. Step here to unlock

4.4 Raising/Lowering the Lift

WARNING! **Risk of Injury**

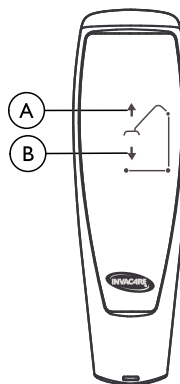
The lift could tip and endanger the patient and assistants.

- Invacare does recommend that the rear casters be left unlocked during lifting procedures to allow the patient lift to stabilize itself when the patient is initially lifted from a chair, bed or any stationary object.

4.4.1 Raising/Lowering an Electric Lift

Refer to the safety information in 4.4 Raising/Lowering the Lift, page 18 before performing this procedure.

1. To raise the lift — Press and hold the UP **A** button to raise the boom and the patient.
2. To lower the lift — Press and hold the DOWN **B** button to lower the boom and the patient.

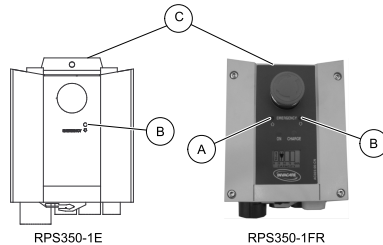
Release the button to stop raising or lowering the lift.

4.5 Activating a Mechanical Emergency Release


There are two types of mechanical emergency release — primary and secondary.

4.5.1 Primary Emergency Release

1. Insert a pen into the hole labeled Emergency Up **(A)** or Emergency Down **(B)** on the control box **(C)**.

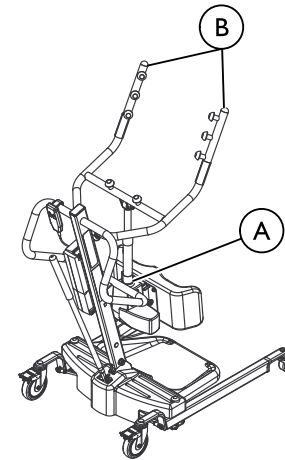4.5.2 Secondary Emergency Release

(A) RED Emergency Grip

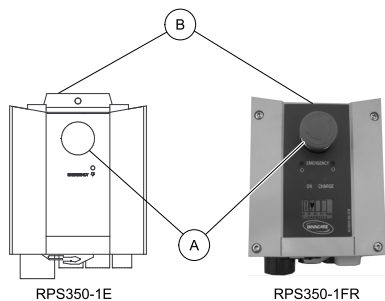
(B) Lift Arms

- i** It is recommended that the primary emergency release be used. The secondary emergency release is only a back-up to the primary emergency release.

In cases where the primary release is either not functioning or unreachable, a secondary emergency release may be used.

1. Pull up on the EMERGENCY grip **(A)** and push down on the lift arms **(B)** at the same time.

4.6 Performing an Emergency Stop

1. Press the RED button **A** on the control box **B** in to stop the boom and patient from raising or lowering.
2. To reset, rotate the emergency button clockwise.

4.7 Charging the Battery

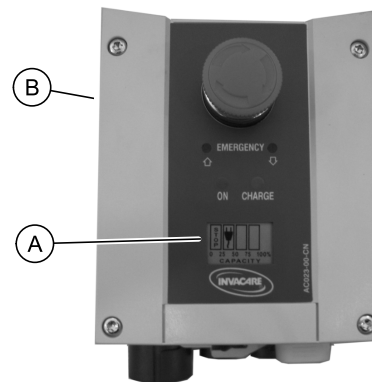
Invacare recommends the battery be recharged daily to prolong battery life.

There are two different methods to charge the battery. One method uses a power cord that attaches to the control box, and the other requires the battery to be mounted to the battery charger. Follow the appropriate procedure to charge the battery for your patient lift.

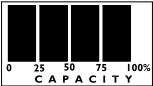
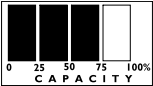
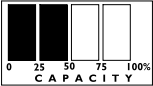
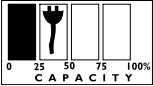
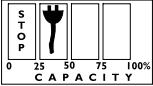
4.7.1 Battery Indicator

This procedure only applies to the RPS350-1FR models.

The battery indicator **A** is located on the control box **B**. The LEDs indicate the battery state:

Control Box Battery Indicator 	Battery State	Description
	Full Charge (100%)	The battery is OK — no need for charging (100%).
	Partial Charge (75%)	The battery is OK — no need for charging (75%).
	Partial Charge (50%)	The battery needs to be charged (50%).
	Low Charge (25%)	The battery needs to be charged (25%). The horn beeps when a button is pressed.
	Low Charge (0%)	<p>The battery needs to be charged.</p> <p>Some of the functionality of the lift is lost and it is only possible to lower the boom.</p> <p>An audible alarm will sound (horn will beep) when battery is low. If the audible alarm sounds during a transfer, complete that transfer and then charge the battery.</p>

4.7.2 Using a Power Cord to Charge the Battery

This procedure only applies to the RPS350-1FR.

Ⓐ = Power Cord Connects Here


CAUTION!

– The emergency stop **MUST NOT** be activated - otherwise it will be impossible to charge the battery. While charging takes place, the patient lift cannot be used. **DO NOT** attempt to move the patient lift without unplugging the power cord from the wall outlet. **DO NOT** attempt to use the patient lift if the battery housing is damaged. Replace a damaged battery housing before any further use.

1. Attach the power cord to the control box.
2. Plug the power cord into a power outlet.
 The battery will charge in approximately 4 hours. Charging must be done in a room with good air ventilation.
3. Disconnect the power cord from the power outlet after the battery has been fully charged.

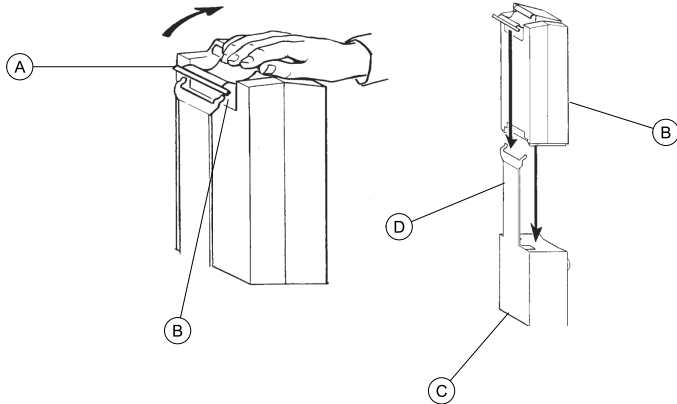4.7.3 Using the Battery Charger to Charge the Battery

 This procedure only applies to the RPS350-1E.

1. Lift up on the handle (A) on the back of the battery (B).
2. Lift the battery up and out away from the control box (C).

CAUTION!

Mounting the battery improperly may cause injury or damage.

- Make sure there is an audible click when mounting battery on the battery charger to confirm proper mounting.

3. Place the battery on the battery charger (D) as shown. Make sure there is an audible click.
 -  The charge LED will illuminate. When charging is complete, charge LED will stop illuminating. A battery needing to be fully recharged will take approximately four hours.
4. Lift up on the handle on the back of the battery.
5. Lift the battery up and out away from the battery charger.

CAUTION!

Mounting the battery improperly may cause injury or damage.

- Make sure there is an audible click when mounting battery on the control box to confirm proper mounting.

6. Reinstall the battery onto the control box as shown. Make sure there is an audible click.
 -  The battery mounts to the control box and battery charger as shown.

5 Lifting the Patient

5.1 Safe Lifting

WARNING!

- DO NOT exceed the maximum weight limitation (SWL) of 159 kg (25 stone). Maximum patient weight is 159 kg (25 stone).
- DO NOT attempt to any transfer without approval of a Healthcare Professional.
- ALWAYS keep hands and fingers clear of moving parts to avoid injury.
- During transfer, with the patient suspended in the sling, DO NOT roll the base of the lift over any uneven surfaces that would cause the lift to become unstable.
- Use the steering handle on the mast assembly at all times to push or pull the stand assist lift.

WARNING!

Operating the Lift and Lifting the Patient

- Ensure equipment used with the stand assist lift is of adequate strength to lift the load (e.g. sling). Ensure that should the equipment malfunction that the person being lifted is not exposed to danger.
- Before positioning the legs of the stand assist lift around the patient, make sure that the patient's feet are out of the way of the foot plate, otherwise injury may occur.

- Adjustments for safety and comfort should be made before moving the patient. Patient's arms should be outside of the sling straps.

WARNING!

Operating the Lift and Lifting the Patient

- Before lifting a patient from a stationary object (wheelchair, commode or bed), slightly raise the patient off the stationary object and check that all sling attachments are secure. If any attachment is not correct, lower the patient and correct the problem, then raise the patient and check again.
- During transfer, with the patient suspended in a sling attached to the lift, DO NOT roll caster base over uneven surfaces that would create an imbalance of the patient lift and could cause the patient lift to tip over. Use steering handle on the mast at ALL times to push or pull the patient lift.
- Invacare recommends locking the rear swivel casters ONLY when positioning or removing the sling (stand assist or transfer) from around the patient.
- Invacare does NOT recommend locking of the rear casters of the patient lift when lifting an individual. Doing so could cause the lift to tip and endanger the patient and assistants. Invacare DOES recommend that the rear casters be left unlocked during lifting procedures to allow the patient lift to stabilize itself when the

patient is initially lifted from a chair, bed or any stationary object.

WARNING!
Using the Sling

- Use the sling that is recommended by a Healthcare Professional for the comfort and safety of the individual that is being lifted.
- Individuals that use the stand assist sling **MUST** be able to support the majority of their own weight, otherwise injury may occur.
- Bleached, torn, cut, frayed or broken slings are unsafe and their use could result in injury or damage. Discard any sling that meets these criteria **IMMEDIATELY**.
- **DO NOT** alter slings.
- Be sure to check the sling attachments each time the sling is removed and replaced, to ensure that it is properly attached before the patient is removed from a stationary object (bed, chair or commode).
- If the patient is in a wheelchair, activate the wheelchair brakes to prevent the chair from moving forwards or backwards.
- Consideration should be taken that prevents exposure to danger to the person being lifted, and that the person can be freed without injury.

WARNING!
Stand Assist Sling

- **DO NOT** use the stand assist sling in combination with the patient lift as a transport device. It is intended to transfer an individual from one resting surface to another (such as a bed to a wheelchair).
- Before lifting the patient, make sure the bottom edge of the stand assist sling is positioned on the lower back of the patient and the patient's arms are outside the stand assist sling. The belt **MUST** be snug, but comfortable on the patient, otherwise the patient can slide out of the sling during transfer possibly causing injury.
- The belt **MUST** be snug, but comfortable on the patient, otherwise the patient can slide out of the sling during transfer, possibly causing injury.

Transfer Stand Assist Sling

- Before lifting the patient, make sure the bottom edge of the transfer sling is at the base of the spine and the patient's arms are outside the transfer sling.
- **DO NOT** raise the patient to a full standing position while using the transfer sling, otherwise injury may occur.

Refer to the patient sling brochure for more information.

5.2 Positioning the Stand Assist Lift

Refer to the Safety section in this manual and review the information in 5.1 Safe Lifting, page 24 before proceeding further and observe all warnings indicated.

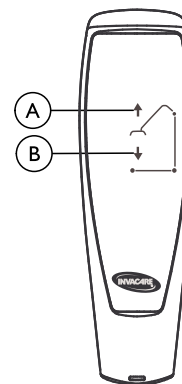
Before positioning the legs of the patient lift under a bed, make sure that the area is clear of any obstructions.

WARNING! **Risk of Injury**

The lift could tip and endanger the patient and assistants.

- The legs of the lift must be in the maximum open position for optimum stability and safety. If it is necessary to close the legs of the lift to maneuver the lift under a bed, close the legs of the lift only as long as it takes to position the lift over the patient and lift the patient off the surface of the bed. When the legs of the lift are no longer under the bed, return the legs of the lift to the maximum open position.

Ⓐ UP Button

Ⓑ DOWN Button

1. Ensure the legs of the stand assist lift are in the maximum open position. If not, use the shifter handle to open the legs.
2. Position the stand assist lift using the mast handle.
3. Press the down arrow button on the hand control to lower the lift arms for easy attachment to the sling.

5.3 Lifting the Patient

WARNING!

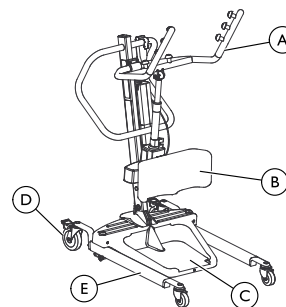
- DO NOT exceed the maximum weight limitation (SWL) of 159 kg (25 stone). Maximum patient weight is 159 kg (25 stone).
- Individuals that use the stand assist sling MUST be able to support the majority of their own weight, otherwise injury may occur.
- DO NOT lock the rear casters of the stand assist lift when lifting an individual. Locking the rear casters could cause the stand assist lift to tip and endanger the patient and assistants. Unlocking the rear casters during lifting procedures allows the lift to stabilize itself while the patient is initially lifted from a wheelchair, bed of any stationary object.
- DO NOT move the patient if the sling is not properly attached to the attachment points on the stand assist lift.
- Make sure that the sling is properly attached BEFORE lifting the patient. If any attachments are not properly in place, correct the problem. When the sling is elevated a few inches/cm off the stationary surface and before moving the patient, check again to make sure the sling and all attachments are securely in place. If any problem is found, lower the patient back onto the stationary surface and correct the problem - otherwise, injury or damage may occur.
- Adjustments for safety and comfort should be made before moving the patient.


WARNING!

- Use the sling that is recommended by the individual's doctor, nurse or medical assistant for the comfort and safety of the individual that is being lifted. Invacare patient slings are made specifically for use with Invacare patient lifts. It may in certain instances, be possible to use other manufacturer's slings on Invacare lifts. Contact your local Invacare office for more information.


Detail A — Stand Assist Lift

Detail B — Sling Attachment

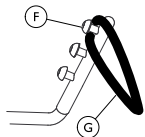Detail C — Brakes

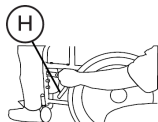Ⓐ Hand Grips	Ⓔ Leg
Ⓑ Knee Pad	Ⓕ Hook
Ⓒ Foot Plate	Ⓖ Loop of sling
Ⓓ Rear Caster	Ⓗ Brake

1. Instruct the patient to hold onto the hand grips on both sides of the stand assist lift (Detail A).
2. Instruct the patient to lean back into the standing or transfer sling.

WARNING!

- Stand Assist Slings - Before lifting the patient, make sure the bottom edge of the sling is positioned on the lower back of the patient and the patient's arms are outside the sling.
- Transfer Stand Assist Slings - Before lifting the patient, make sure the bottom edge of the sling is at the base of the spine and the patient's arms are outside the sling.

Refer to the patient sling user manual for more information.

The patient **MUST** be in a seated position first. Use the head section of the bed to get the patient upright then move legs over the side of the bed.

3. Ensure the following:
 - a. The patient's knees are secure against the knee support.
 - b. The patient's feet are properly positioned on the foot plate.
 - c. Slings:
 - the bottom edge of the stand assist sling is positioned on the patient's lower back.
 - the bottom edge of the transfer stand assist sling is at the base of the patient's spine.
 - the patient's arms are outside the sling.
 - the loops of the sling are completely on the hooks of the lift arms (Detail B).
 - d. The rear casters are unlocked.
 - e. The legs are in the maximum open position.

**WARNING!**

– If transferring a patient from a wheelchair, the wheelchair brakes **MUST** be in the locked position before lowering the patient into the wheelchair. Otherwise, injury may occur.

4. If transferring from a wheelchair, engage the brakes on the wheelchair (Detail C).
5. Press the UP arrow button on the hand control to raise the patient above the surface (bed, wheelchair or commode). The patient's weight will be fully supported by the stand assist lift.

The lower center of gravity provides stability making the patient feel more secure and the lift easier to move.

5.4 Moving the Patient

**WARNING!**

– The legs of the stand assist lift **MUST** be in the maximum open position for optimum stability and safety. If the patient is in a sling and it becomes necessary to move through a narrow passage, close the legs of the stand assist lift only as long as it takes to move through the passage. When the stand assist lift is through the passage, return the legs to the maximum open position.

– **DO NOT**, during transfer of a patient suspended in the lift sling, roll caster base over uneven surfaces that would create an imbalance of the lift. This could cause the lift to tip over. Use the mast handle at all times to push or pull the lift.

1. Ensure the legs of the stand assist lift are in the maximum open position. If not, press the OPEN LEGS button on the hand control to move the legs to the maximum open position.
2. Move the stand assist lift away from the surface they were lifted from.
3. Slowly move the patient to the desired surface.

6 Transferring the Patient

6.1 Safe Transfer

**WARNING!**

- DO NOT exceed the maximum weight limitation (SWL) of 159 kg (25 stone). Maximum patient weight is 159 kg (25 stone).
- DO NOT attempt to transfer a patient without approval of a Healthcare Professional. Thoroughly read the instructions in this owner's manual, observe a trained team of experts performing the lifting procedures and then perform the entire lift procedure several times with proper supervision and a capable individual acting as a patient. Training can be provided. Contact your local Invacare office for more information.
- DO NOT move the patient if the sling is not properly connected to the attachment points of the stand assist lift. Check that the sling is properly connected to the attachment points prior to lifting a patient. If any attachments are not properly in place, correct the problem. When the sling is elevated a few inches/cm off the stationary surface and before moving the patient, check again to make sure that all sling attachments are secure. If any attachments are not properly in place, lower the patient back onto the stationary surface and correct this problem - otherwise, injury or damage may occur.

- Adjustments for safety and comfort should be made before moving the patient. The patient's arms should be outside the straps.

**WARNING!**

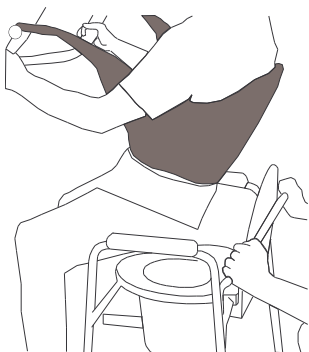
- Use the sling that is recommended by a Healthcare Professional for the comfort and safety of the individual that is being lifted.
- DO NOT lock the rear casters of the stand assist lift when lifting an individual. Locking the rear casters could cause the stand assist lift to tip and endanger the patient and assistants.
- The legs of the stand assist lift **MUST** be in the maximum open position for optimum stability and safety. If the patient is in a sling and it becomes necessary to move through a narrow passage, close the legs of the stand assist lift only as long as it takes to move through the passage. When the stand assist lift is through the passage, return the legs to the maximum open position. If it is necessary to close the legs to maneuver the stand assist lift under a bed, close the legs only as long as it takes to position the stand assist lift over the patient and lift the patient off the surface of the bed. When the legs of the stand assist lift are no longer under the bed, return the legs to the maximum open position.
- Be sure to check the sling attachments each time the sling is removed and replaced to

ensure that it is properly attached before the patient is removed from a surface.


The use of one assistant is based on the evaluation of the health care professional for each individual case.

6.2 Transferring to a Commode


Detail A

Detail B

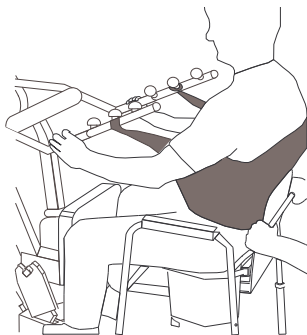Detail C

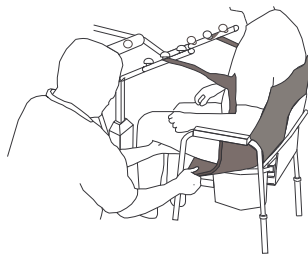WARNING!

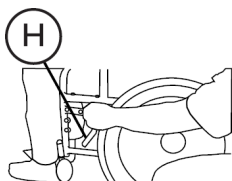
– Invacare recommends locking the rear swivel casters only when positioning or removing the sling from around the patient.

1. Lift the patient from the side of the bed.
 2. Press the UP button on the hand control to elevate the patient high enough to clear the arms of the commode chair. Their weight will be supported by the stand assist lift.
 3. Guide the patient onto the commode chair. This may require two assistants.
 4. Press the down arrow button to lower the patient onto the commode chair.
 5. Lock the rear swivel casters on the stand assist lift.
 6. Depending on the style of sling, perform one of the following:
 - Stand Assist Sling - unhook the stand assist sling from the attachment points on the stand assist lift.
 - Transfer Stand Assist Sling -
 - a. Unhook the transfer stand assist sling from the bottom attachment points on the stand assist lift.
 - b. Lift up on the patient's legs and remove the thigh supports from underneath the patient.
 - c. If desired, unhook the transfer stand assist sling from the top attachment points on the stand assist lift.
-  The patient can remain in the upper portion of the transfer stand assist sling while using the commode.

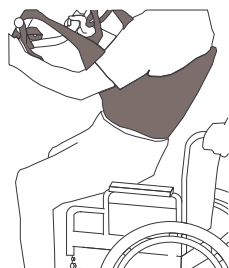
7. Instruct or assist the patient in lifting their feet off the foot plate.
8. Remove the sling from around the patient.
9. Unlock the rear casters and pull the stand assist lift away from the commode.
10. When complete, recheck the sling for correct attachments.

6.3 Transferring to a Wheelchair

Detail A

Detail B

Detail C

  = Wheelchair Brake

1. Ensure the legs of the lift with the patient in the sling are in the open position. Press the legs open button until in maximum open position.
2. Move the wheelchair into position.
3. Engage the brakes of the wheelchair to prevent movement of the chair (Detail A).

WARNING!

– DO NOT place the patient in the wheelchair if the brakes are not engaged. The wheelchair brakes MUST be in a locked position before lowering the patient into the wheelchair for transport. Otherwise, injury may result.

4. Position the patient over the wheelchair (Detail “B”).
5. Press the down arrow button and lower the patient into the wheelchair.

WARNING!

– Lock the rear swivel casters ONLY when positioning or removing the sling (stand assist or transfer stand assist) from around the patient. Otherwise, injury may occur.

6. Lock the rear swivel casters.
7. Unhook the sling from all attachment points on the lift (Detail C).
8. Instruct patient to lift their feet off the foot plate. Assist the patient if necessary.

9. Remove the sling from around the patient.
10. Unlock the rear casters and pull the stand assist lift away from the wheelchair.

6.4 Transferring to a Bed

The lower center of gravity provides stability making the patient feel more secure and the lift easier to move.

1. Position the patient as far over the bed as possible.
 - If patient is being transferred from a surface that is lower than the bed, press the up arrow button to raise the patient above the surface of the bed. The patient should be elevated just high enough to clear the bed with their weight fully supported by the lift.
2. Press the down arrow button and lower the patient onto the bed.

WARNING!

– Invacare recommends locking the rear swivel casters **ONLY** when positioning or removing the sling from around the patient.

3. Lock the rear swivel casters.
4. Unhook the stand assist or transfer stand assist sling from all attachment points on the stand assist lift.
5. Instruct the patient to lift their feet off of the foot plate.
 - Assist the patient if necessary.

6. Remove the stand assist or transfer stand assist from around the patient.
7. Unlock the rear casters and pull the stand assist lift away from the bed.

7 Troubleshooting

7.1 Troubleshooting Table

SYMPTOMS	FAULTS	SOLUTION
Noisy or dry sound from pivots.	Needs lubrication.	Refer to Lubricating the Lift .
Electric actuator fails to lift when button is pressed.	Hand-control or actuator connector loose.	Connect hand control or actuator connector. Ensure connectors are seated properly and fully connected.
	Battery low.	Charge batteries. Refer to 4.7 Charging the Battery, page 20.
	RED emergency stop button pressed IN.	Rotate RED emergency stop button CLOCKWISE until it pops out.
	Battery not connected properly to control box.	Reconnect the battery to the control box. Refer to 4.7 Charging the Battery, page 20.
	The connecting terminals are damaged.	Replace the battery pack. Refer to 4.7 Charging the Battery, page 20.
	Electric actuator in need of service or load is too high.	Refer to 8.7 Replacing the Electric Actuator, page 41. Contact your Dealer or Invacare representative.
Unusual noise from lift arm actuator.	Actuator is worn or damaged or spindle is bent.	Refer to 8.7 Replacing the Electric Actuator, page 41. Contact your Dealer or Invacare representative.
Lift arms will not lower in uppermost position.	Lift arms require a minimum weight load to lower from the uppermost position.	Pull down slightly on the lift arms.
Legs do not open and close properly.	Base may require adjustment.	Refer to 8.6 Adjusting the Base, page 40.

 If problems are not remedied by the suggested means, please contact your dealer or Invacare.

8 Maintenance

8.1 Maintenance Safety Inspection Checklist

WARNING!

- Maintenance **MUST** be performed **ONLY** by qualified personnel.
- A competent person must ensure that the strength and stability of the product remains adequate for all tasks if parts are replaced.
- **DO NOT** overtighten the mounting hardware, this will damage the mounting brackets.

Follow the maintenance procedures described in this manual to keep your patient lift in continuous service.

The Invacare Patient Lift is designed to provide a maximum of safe, efficient and satisfactory service with minimum care and maintenance.

All parts of the Invacare Lift are made of the best grades of steel, but metal to metal contact will wear after considerable use.

There is no adjustment or maintenance of the casters, other than cleaning, lubrication and checking axle and swivel bolts for tightness. Remove all debris, etc. from the wheel and swivel bearings. If any parts are worn, replace these parts **IMMEDIATELY**.

If you question the safety of any part of the lift, contact your Dealer **IMMEDIATELY**.

The UK Health and Safety Executive's Lifting Operations and Lifting Equipment Regulations 1998 require any equipment that is used in the workplace to lift a load be subject to safety inspection on a six monthly basis. This is to be carried out by a suitably trained individual. Please refer to the HSE website for guidance (www.hse.gov.uk).

Regular maintenance of patients lifts and accessories is necessary to assure proper operation. It is the responsibility of the person responsible for the patient lift to ensure that this maintenance is carried out.

After the first 12 months of operation, inspect all pivot points and fasteners for wear. If the metal is worn, the parts **MUST** be replaced. Perform this inspection every six months thereafter.

Date of Inspection:	Initials:
THE CASTER BASE <ul style="list-style-type: none"> <input type="checkbox"/> Inspect for missing hardware. <input type="checkbox"/> Base opens/closes with ease. <input type="checkbox"/> Inspect casters and axle bolts for tightness. <input type="checkbox"/> Inspect casters for smooth swivel and roll. <input type="checkbox"/> Inspect and clear wheels of debris. <input type="checkbox"/> Inspect pivot joints for wear. 	
SLINGS AND HARDWARE <ul style="list-style-type: none"> <input type="checkbox"/> Check all sling attachments each time it is used to ensure proper connection and patient safety. <input type="checkbox"/> Inspect sling material for wear. <input type="checkbox"/> Inspect straps for wear. <input type="checkbox"/> Inspect stitching. 	
THE ELECTRIC ACTUATOR ASSEMBLY <ul style="list-style-type: none"> <input type="checkbox"/> Check for leakage. <input type="checkbox"/> Inspect hardware on mast, boom and base. <input type="checkbox"/> Check for wear or deterioration. If damaged, return to factory. <input type="checkbox"/> Cycle to ensure smooth quiet operation of the electric actuator. 	

THE LIFT ARMS AND LINKAGE

- Check all hardware and attachment points.
- Inspect for bends or deflections.
- Inspect bolted joints of the lift arms for wear.
- Inspect to ensure that the lift arms are centered between the base legs.
- Inspect pivot joints for wear.
- Check sling hooks for wear or deflection.

THE MAST

- Mast must be securely assembled to lift arms.
- Inspect for bends or deflections.
- Inspect pivot joints for wear.

CLEANING

- Whenever necessary.

8.2 Cleaning the Sling and the Lift

WARNING!

- After EACH laundering (in accordance with laundering instructions on the sling), inspect the sling for wear, tears and loose stitching. Discard any patient sling that meets these criteria IMMEDIATELY.

Cleaning the Sling

The sling should be washed regularly in water temperature not exceeding 203°F (95°C) and a biological solution. Refer to the washing instructions in the sling user manual.

Cleaning and Disinfecting the Lift

To prevent cross-infection, the hoist must be cleaned and disinfected after each use.

A soft cloth, dampened with water and a small amount of mild detergent, is all that is needed to clean the patient lift. The lift can be cleaned with non-abrasive cleaners.

Never use acids, alkaline or solvents for cleaning the lift. Dry the lift carefully after cleaning.

Motors, control unit and mounting parts can be damaged if the lift is cleaned any other way than stated above.

The lift must be wiped with a moistened, firmly wrung cloth with ordinary household disinfectants. Only use disinfection detergents approved by the facility.

8.3 Reuse

This product is suitable for reuse. The maximum number of times it can be reused is dependent upon product condition.

To prevent the transmission of infection, the patient lift and slings must be cleaned after each use. Before reuse, refer to Cleaning the Sling and the Lift .

8.4 Detecting Wear and Damage

It is important to inspect all stressed parts, such as slings, lifting arm and any pivot for slings for signs of cracking, fraying, deformation or deterioration. Replace any defective parts IMMEDIATELY and ensure that the lift is not used until repairs are made.

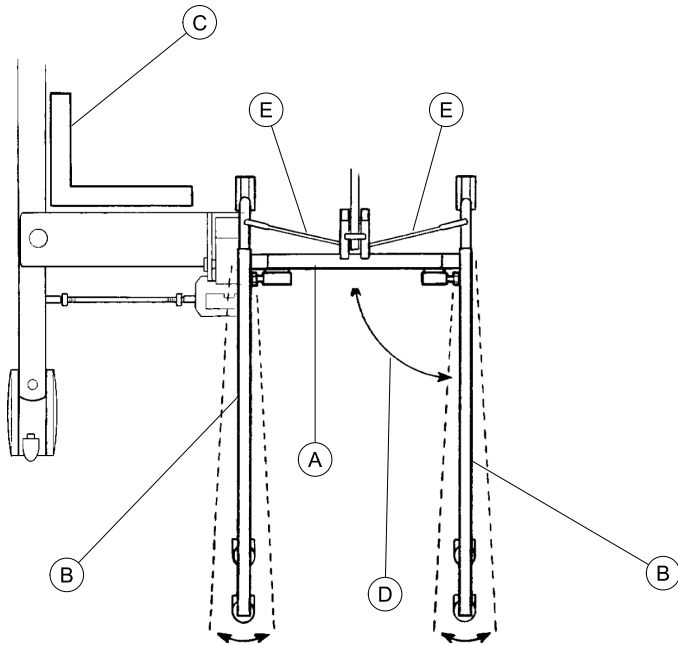
8.5 Lubricating the Lift

The Invacare lift is designed for minimum maintenance. However, a six month check and lubrication should ensure continued safety and reliability.

Keep lift and slings clean and in good working order. Any defect should be noted and reported to your dealer as soon as possible.

The casters MUST swivel and roll smoothly. A light grease (waterproof auto lubricant) may be applied to the ball bearing swivel of the casters once a year. Apply more frequently if the casters are exposed to extreme moist conditions.

8.6 Adjusting the Base


The base (A) adjustment should not require any attention other than adjusting the linkage rods.

- ! Damage to the lift may occur.
 - The legs must be in the fully closed position before adjusting the linkage rods.

1. Close the legs all the way. Refer to 4.2.1 Closing/Opening Manual Legs, page 17.
2. Check the squareness of the legs (B) when in the closed position.
3. Place a square (C) on the inside of the legs and base to determine the 90° alignment (D).
4. Adjust the linkage rods (E) until 90° alignment is achieved.

- ! Damage to the lift may occur.
 - DO NOT adjust the linkage rods so the legs are at less than 90° alignment.

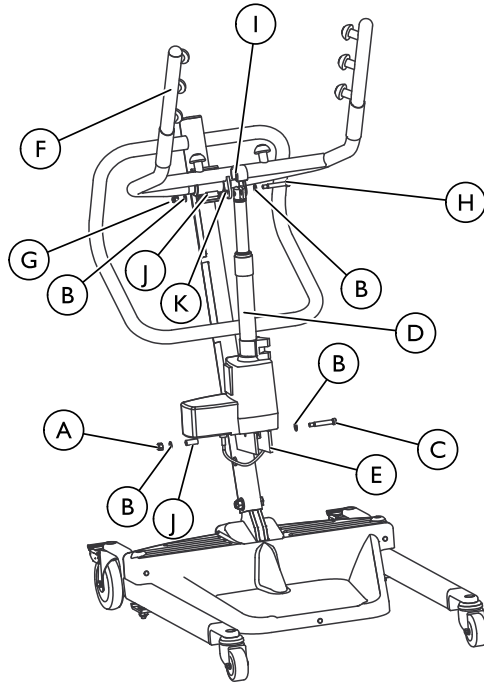8.7 Replacing the Electric Actuator

(A) Bottom nut	(G) Top nut
(B) Washer	(H) Bolt
(C) Shoulder bolt	(I) Bracket
(D) Electric actuator	(J) Bushing

(E) Mast mounting bracket	(K) Lift arm mounting bracket
(F) Lift arm	

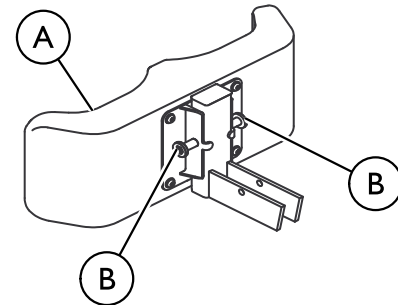
WARNING!

– DO NOT overtighten the mounting hardware. This will damage mounting brackets.

1. Remove the bottom nut, washers, shoulder bolt and bushing that secure the electric actuator to the mast mounting bracket.
2. Rest the lift arm on your shoulder and remove the top nut, bolt, bracket, bushing and washers from the lift arm mounting bracket.
3. Remove the electric actuator.
4. Reverse STEPS 1–3 to install the new electric actuator.

8.8 Adjusting the Knee Pad Height

(A) Knee pad	(B) Adjustment pins
--------------	---------------------

WARNING!

- NEVER adjust the knee pad while patient is in the standing position.
- NEVER try to adjust the knee pad while the lift is moving.
- ALWAYS make sure that the adjustment pins are engaged in corresponding height adjustment holes before use.

1. Pick a height setting that will be comfortable to the patient and provide the necessary support.
 -  The knee pad should be positioned so that the knee portion of the leg is in contact with the pad.
2. Using both hands, pull both adjustment pins outward at the same time.
3. Position the knee pad to the desired height and release adjustment pins into the corresponding alignment holes.
4. Check to make sure that both pins are engaged.

9 Technical data

9.1 RPS3501E and RPS3501FR Stand Assist Patient Lift

Height at Sling Hook-up — MAX	168 cm
Height at Sling Hook-up — MIN	102 cm
Base Width (open)	94 cm
Base Width (closed)	66 cm
Base height (clearance)	11,5 cm
Base length	90 cm
Overall height	125 cm
Overall length	99 cm
Overall width	65,5 cm
Castor size (FRONT/REAR)	8 cm / 13 cm
Sling Options	Stand Assist or Transfer Stand Assist
Sling Material	Polyester
Maximum weight limitation (SWL) (patient + sling + lift arms)	159 kg (25 stone)
Maximum patient weight	159 kg (25 stone)
Total weight (weight out of carton)	49 kg (8 stone)

Battery (voltage output)	24V DC mb max. 240VA (RCHBL)
Charger Input (voltage supply)	100-240V AC ~ 50/60 Hz
Charger Output/Charging Time	29.5V DC 2.9 Ah Max 6 hours
Audio/Visual Low Battery Alarm	Yes
Motor Safety Devices	Anti-Entrapment
*Approx. Lifts per Charge (working ability)	*100-200 Cycles per charge
Max. current input	Max. 400 mA
Operating temperature	5° to 40° C
Air humidity	20% to 90% at 30 not condensing
Sound pressure	< 50 dB
Atmospheric pressure	700 to 1060 hPa
Storage temperature	above 0° C
Storage air humidity	less than 60%
Storage atmospheric pressure	700 hPa to 1060 hPa
Degree of protection, control unit	IPX4

Degree of protection, hand control	IPX4
Degree of protection, motor	IPX4
Insulation class	Class II equipment, Type B applied part
Intermittens	10%, max, 2 minutes/18 minutes
Battery capacity	2,9 Ah
Manual emergency lowering	Yes
Electric emergency lowering/lifting	Yes/Yes
Expected lifetime	8 years
Lifting Speed	The lift passes the “velocity of lifting and lowering” in the ENISO10535 (<0.15 m/s under maximum load and <0.25 m/s unloaded)

 * Varies depending on load and stroke.

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Invacare Limited
Equipment (Suzhou) Co.,Ltd.