Aquatec® Disk for Dot

en	Turntable User Manual	1
de	Drehbare Sitzfläche Gebrauchsanweisung	2
fr	Plateau tournant Manuel d'utilisation	3
it	Piastra girevole Manuale d'uso	4
es	Plato Manual del usuario	4
pt	Giratório Manual de utilização	5
nl	Draaischijf Gebruiksaanwijzing	5
da	Drejeanordning Brugsanvisning	6
no	Dreieskive Bruksanvisning	7
sv	Svängskiva Bruksanvisning	7
fi	Kääntölevy Käyttöohje	8

en

I General

I.1 Images

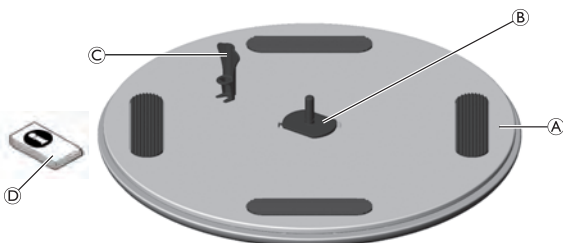
Fig. 1

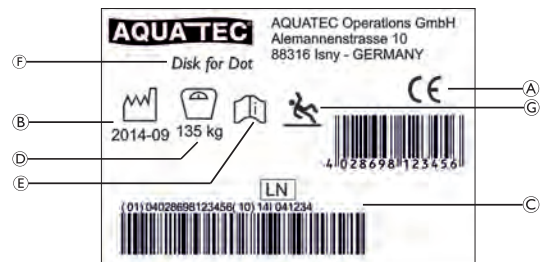
Fig. 2

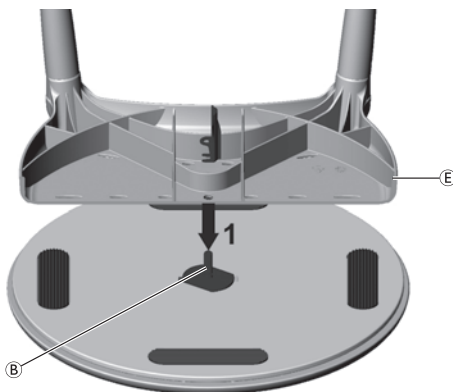
Fig. 3

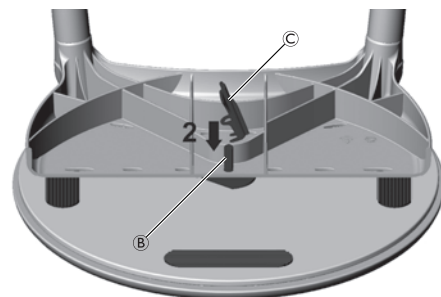
Fig. 4

This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

AQUATEC®

Yes, you can.®

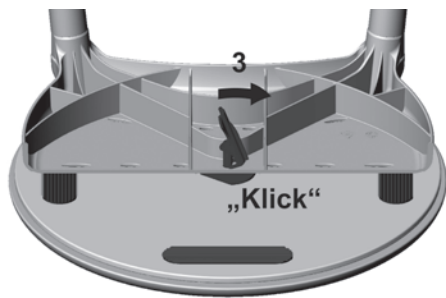
Fig. 5

Fig. 6

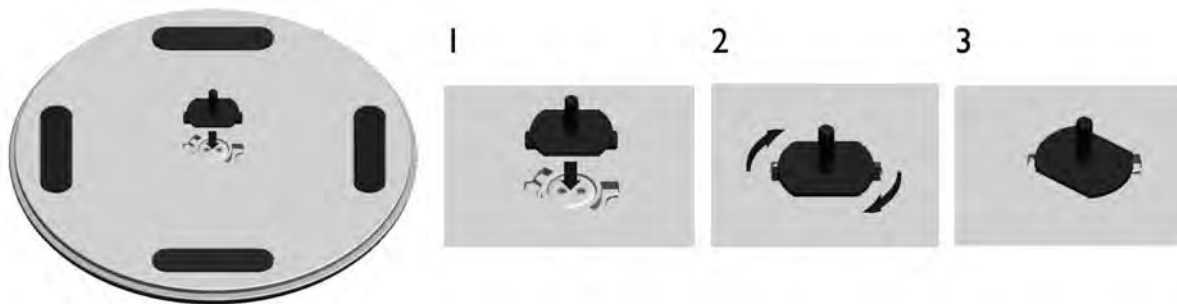
Fig. 7

1.2 Intended use

IMPORTANT!

– Read sections General, Safety, Maintenance and After use contained in the Aquatec® Dot shower stool User Manual in conjunction with this document.

The Aquatec® Disk is to be used solely in combination with the Aquatec® Dot, to provide a turnable seating surface. The combined product is intended to be used as an aid for sitting in the bathroom or in the shower for people with sufficient torso stability. Any other use is prohibited.

Operating temperature: 10 – 40 °C

1.3 Scope of delivery (Fig. 1)

The following components are included in the scope of delivery:

Ⓐ	Turnable disk
Ⓑ	Adapter pin (pre-mounted)
Ⓒ	Fastening clip
Ⓓ	User manual

1.4 Service life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

1.5 Labels and symbols on the product (Fig. 2)

Ⓐ	Conformity mark
Ⓑ	Date of manufacture
Ⓒ	Lot number
Ⓓ	Load capacity
Ⓔ	Note (observe accompanying documents)

Ⓕ	Product designation
Ⓖ	Warning symbol (risk of slipping)

The identification label is attached to the underside of the disk.

2 Setup and usage

2.1 Assembling the product

1. Attach the pre-assembled shower stool Aquatec® Dot Ⓔ with its central hole on the adapter pin Ⓑ of the Aquatec® Disk Ⓐ (Fig. 3).
2. Slide the fastening clip Ⓒ with its hole over the adapter pin Ⓑ of the Aquatec® Disk (Fig. 4).
3. Lock the fastening clip by pressing it sideways (Fig. 5).

WARNING!

Risk of injury

– Make sure that the fastening clip fully engages with an audible “click” with the adapter pin.

2.2 Attaching the adapter (Fig. 7)

The adapter Ⓑ can be replaced in case of damage or wear and is available as a spare part from your specialized dealer.

1. Attach the adapter to the Disk with both nibs positioned in the cut out of the disk Ⓐ.
2. Turn the adapter 30° clockwise using a 35 mm open wrench, if present.
3. Check that the adapter is fully engaged.

2.3 Using the product

1. Carefully sit down on the shower stool.
2. Rotate into the desired position. (Fig. 6)

3 Technical data

∅	365 mm
Height	25 mm

Product weight	1.6 kg
Max. user weight	135 kg
Materials*	PP LGF

*All product components are corrosion resistant and latex free.

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Invacare AB
Algol-Trehab Oy
Aquatec Operations GmbH